

LIFESAVER USER MANUAL

1.

PUT ON WITH ONE HAND



or, where it is not possible to place a ready-to-puff tourniquet

SLIDE the injured limb through the loop
POSITION the tourniquet approx. 5-7 cm above the bleeding site.

2.



PULL the tie as tightly as possible.

3.



TWIST the windlass bar until bleeding stops.

4.



PLACE the windlass bar in the clip until you hear a "CLICK".

5.



SECURE the loose end of the strap with Velcro.

6.



WRITE the time of tourniquet application – with the included marker, on white strap (there are two straps).

PUT ON WITH BOTH HANDS



WRAP the LifeSaver tourniquet around the limb,
PASS the end of the strap through the buckle,
PLACE the tourniquet approx. 5-7 cm above the bleeding site.

General guidelines

In order to effectively stop bleeding, the LifeSaver Tourniquet should be applied directly to the skin at a distance of approx. 5-7 cm above the bleeding site. In the event that the bleeding site is not visible, the tourniquet should be placed on clothing, high on the limb. However, remember to avoid the application of tourniquet on metal items of clothing to prevent additional injury.

Disclaimer

It is important for the user to remember that the manufacturer of LifeSaver Tourniquet is not responsible for any injuries or damage resulting from its use. The use of a tourniquet in accordance with rescue guidelines allows to effectively stop bleeding and minimize the risk of more serious injuries.

